

# Kimberly-Clark\*

## MIC-KEY\* Bolus Extension Set with Secur-Lok\* Connector

### MIC-KEY\* BOLUS EXTENSION SET WITH CATH TIP, SECUR-LOK\* STRAIGHT CONNECTOR AND CLAMP

#### DIRECTIONS FOR USE

1. Remove the plug from the MIC-KEY\* feeding tube to reveal the MIC\* feeding tube anti-reflux valve.
2. Insert the Bolus Extension Set connector into the MIC-KEY\* feeding port. Align the black marks on both the Bolus Extension Set and the MIC-KEY\*, twist into place with a CLOCKWISE half turn.
3. Flush the tube with water, (10 ml for adults, 3-5 ml for children) to insure it is free of clogs.
4. Administer the feeding.
5. After feeding, flush the Bolus Extension Set and the MIC-KEY\* with 5 to 10 ml of warm water.
6. Disconnect the Bolus Extension Set from the MIC-KEY\* by rotating the feeding set connector COUNTER-CLOCKWISE until it stops. Gently remove the connector from the MIC-KEY\*. Cap the MIC-KEY\*.
7. Wash the Bolus Extension Set with warm, soapy water and rinse thoroughly.



0123-12  
0123-24

#### Single Patient Use Only.

**WARNING: FOR ENTERAL NUTRITION AND/OR MEDICATION ONLY.**



Use By

**REF**  
Reorder  
Number

**LOT**

Lot  
Number

Quantity



Non sterile

**CE** 0344

\* Registered Trademark or Trademark of Kimberly-Clark Worldwide, Inc. or its affiliates.  
© 2006 KCWW. All rights reserved.

R111985 1/06



DEHP (Di(2-ethylhexyl) Phthalate)  
Free Formulation



Read  
Instructions

Rx Only  
(USA)



Single Use

Manufactured by Ballard Medical Products, Draper, Utah 84020 USA  
Distributed in the U.S. by Kimberly-Clark Global Sales, Inc. Roswell, GA. 30076 USA  
Kimberly-Clark N.V., Belgicastraat 13, 1930 Zaventem, Belgium  
<http://www.kchealthcare.com> In USA call 1-800-528-5591; International call + 1 801 572 6800