


Kimberly-Clark

MIC-KEY* Extension Set with Secur-Lok* Connector

MIC-KEY* EXTENSION SET WITH SECUR-LOK* RIGHT ANGLE CONNECTOR AND 2 PORT "Y" AND CLAMP

DIRECTIONS FOR USE



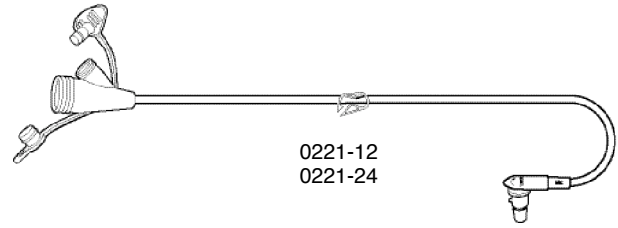








1. Remove the plug from the MIC-KEY* feeding tube to reveal the MIC* feeding tube anti-reflux valve.
2. Connect the Extension Set by aligning the black line on the connector with the black line on the MIC-KEY*. Lock into place by rotating it CLOCKWISE 3/4 turn.
3. Flush the Extension Set and the MIC-KEY* with warm water (10 ml for adults, 3–5 ml for children). Check for ease of flushing.
4. Open the feeding port to perform gastric decompression or enteral feeding.
5. After feeding or decompression, flush the Extension Set and the MIC-KEY* with 5–10 ml of warm water.
6. To disconnect the Extension Set from the MIC-KEY*, rotate the extension set connector COUNTER-CLOCKWISE until the black line on the MIC-KEY* aligns with black line on the Extension Set. Gently remove the connector from the MIC-KEY*. Replace the plug into the MIC-KEY*.
7. After each use, wash the Extension Set with warm, soapy water and rinse thoroughly.



SINGLE PATIENT USE ONLY.

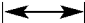WARNING: FOR ENTERAL NUTRITION AND/OR MEDICATION ONLY.

Contains DEHP (see www.kchealthcare.com/DEHP for details)

 Quantity of Units in Dispenser	 Date of Manufacture	 Do Not Use If Package Damaged	REF Reorder Number
 Single Use	 Read Instructions	 Diameter	STERILE R Sterilized by Gamma Irradiation
 Length	LOT Lot Number	Rx Only (USA)	 Use By
 Quantity of Units in Case	 DEHP (Di(2-ethylhexyl) Phthalate) Free Formulation		

* Registered Trademark or Trademark of Kimberly-Clark Worldwide, Inc. or its affiliates. © 2003 KCWW. All rights reserved.

Manufactured by Ballard Medical Products, Draper, Utah 84020 USA
Distributed in the U.S. by Kimberly-Clark Global Sales, Inc. Roswell, GA. 30076 USA
Kimberly-Clark N.V., Belgicastraat 17, 1930 Zaventem, Belgium
<http://www.kchealthcare.com> In USA call 1-800-528-5591; International call + 1 801 572 6800