


Kimberly-Clark

MIC-KEY* Medication Set with Secur-Lok* Connector

MIC-KEY* MEDICATION SET WITH SECUR-LOK* RIGHT ANGLE CONNECTOR AND 2 PORT "Y"

DIRECTIONS FOR USE



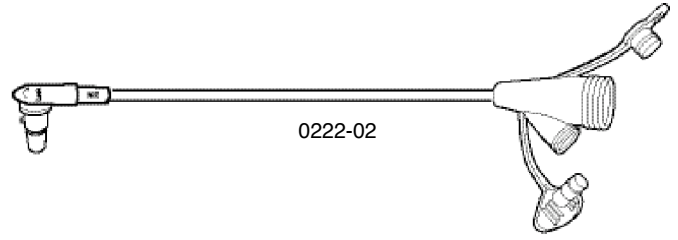








1. Remove the plug from the MIC-KEY* feeding tube to reveal the MIC* feeding tube anti-reflux valve.
2. Snap the Medication Set into place by aligning the black line on the Medication Set with the black line on the MIC-KEY*. Lock into place by rotating it CLOCKWISE 3/4 turn.
3. Flush the Medication Set and the MIC-KEY* with 1 ml to 2 ml of warm water. Check for ease of flushing before medication administration.
4. After medication administration, flush the Medication Set and the MIC-KEY* with 1 ml to 2 ml of warm water.
5. To disconnect the Medication Set from the MIC-KEY* rotate the Medication Set COUNTER-CLOCKWISE until the black lines on the Medication Set and the MIC-KEY* are aligned. Gently remove the Medication Set from the MIC-KEY*. Replace the plug into the MIC-KEY*.
6. After each use, wash the Medication Set with warm, soapy water and rinse thoroughly.



SINGLE PATIENT USE ONLY.

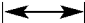WARNING: FOR ENTERAL NUTRITION AND/OR MEDICATION ONLY.

Contains DEHP (see www.kchealthcare.com/DEHP for details)

 Quantity of Units in Dispenser	 Date of Manufacture	 Do Not Use If Package Damaged	REF Reorder Number
 Single Use	 Read Instructions	 Diameter	STERILE R Sterilized by Gamma Irradiation
 Length	LOT Lot Number	Rx Only (USA)	 Use By
 Quantity of Units in Case	 DEHP (Di(2-ethylhexyl) Phthalate) Free Formulation		

* Registered Trademark or Trademark of Kimberly-Clark Worldwide, Inc. or its affiliates. © 2003 KCWW. All rights reserved.

Manufactured by Ballard Medical Products, Draper, Utah 84020 USA
 Distributed in the U.S. by Kimberly-Clark Global Sales, Inc. Roswell, GA. 30076 USA
 Kimberly-Clark N.V., Belgicastraat 17, 1930 Zaventem, Belgium
<http://www.kchealthcare.com> In USA call 1-800-528-5591; International call + 1 801 572 6800