

# Kimberly-Clark

## MIC-KEY\* Bolus Extension Set with Secur-Lok\* Connector

MIC-KEY\* BOLUS EXTENSION SET WITH CATH TIP, SECUR-LOK\* STRAIGHT CONNECTOR AND CLAMP

### DIRECTIONS FOR USE

1. Remove the plug from the MIC-KEY\* feeding tube to reveal the MIC\* feeding tube anti-reflux valve.
2. Insert the Bolus Extension Set connector into the MIC-KEY\* feeding port. Align the black marks on both the Bolus Extension Set and the MIC-KEY\*, twist into place with a clockwise half turn.
3. Flush the tube with water, (10 ml for adults, 3–5 ml for children) to insure it is free of clogs.
4. Administer the feeding.
5. After feeding, flush the Bolus Extension Set and the MIC-KEY\* with 5 to 10 ml of warm water.
6. Disconnect the Bolus Extension Set from the MIC-KEY\* by rotating the feeding set connector COUNTER-CLOCKWISE until it stops. Gently remove the connector from the MIC-KEY\*. Cap the MIC-KEY\*.
7. Wash the Bolus Extension Set with warm, soapy water and rinse thoroughly.














0223-12  
0223-24

**SINGLE PATIENT USE ONLY.**

**WARNING: FOR ENTERAL NUTRITION AND/OR MEDICATION ONLY.**

Contains DEHP (see [www.kchealthcare.com/DEHP](http://www.kchealthcare.com/DEHP) for details)

 Quantity of Units in Dispenser	 Date of Manufacture	 Do Not Use If Package Damaged	REF Reorder Number
 Single Use	 Read Instructions	 Diameter	STERILE   R Sterilized by Gamma Irradiation
 Length	LOT Lot Number	Rx Only (USA)	 Use By
 Quantity of Units in Case	 DEHP (Di(2-ethylhexyl) Phthalate) Free Formulation		

\* Registered Trademark or Trademark of Kimberly-Clark Worldwide, Inc. or its affiliates. © 2003 KCWW. All rights reserved.

Manufactured by Ballard Medical Products, Draper, Utah 84020 USA  
Distributed in the U.S. by Kimberly-Clark Global Sales, Inc. Roswell, GA. 30076 USA  
Kimberly-Clark N.V., Belgicastraat 17, 1930 Zaventem, Belgium  
<http://www.kchealthcare.com> In USA call 1-800-528-5591; International call + 1 801 572 6800