

Kimberly-Clark

MIC-KEY* Threaded Extension Set with Secur-Lok* Connector

MIC-KEY* THREADED EXTENSION SET WITH SECUR-LOK* RIGHT ANGLE CONNECTOR FOR MIC-KEY* LOW PROFILE GASTROSTOMY TUBES

DIRECTIONS FOR USE

1. Remove the plug from the MIC-KEY* feeding tube to reveal the MIC* feeding tube anti-reflux valve.
2. For gastric decompression or enteral feeding, connect the SECUR-LOK* Extension Set connector by aligning the black line on the connector with the black line on the MIC-KEY*. Lock into place by rotating it CLOCKWISE 3/4 turn.
3. Flush the SECUR-LOK* Extension Set and the MIC-KEY* with warm water (10 ml for adults, 3–5 ml for children). Check for ease of flushing.
4. After feeding or decompression, flush the SECUR-LOK* Extension Set and the MIC-KEY* with 5–10 ml of warm water.
5. To disconnect the SECUR-LOK* Extension Set from the MIC-KEY* rotate the extension set connector COUNTER-CLOCKWISE until the black line on the MIC-KEY* aligns with black line on the Extension Set. Gently remove the connector from the MIC-KEY*. Replace the plug into the MIC-KEY*.
6. After each use, wash the SECUR-LOK* Extension Set with warm, soapy water and rinse thoroughly.














0226-12
0226-24

SINGLE PATIENT USE ONLY.

WARNING: FOR ENTERAL NUTRITION AND/OR MEDICATION ONLY.

Contains DEHP (see www.kchealthcare.com/DEHP for details)

 Quantity of Units in Dispenser	 Date of Manufacture	 Do Not Use If Package Damaged	REF Reorder Number
 Single Use	 Read Instructions	 Diameter	STERILE R Sterilized by Gamma Irradiation
 Length	LOT Lot Number	Rx Only (USA)	 Use By
 Quantity of Units in Case	 DEHP (Di(2-ethylhexyl) Phthalate) Free Formulation		

* Registered Trademark or Trademark of Kimberly-Clark Worldwide, Inc. or its affiliates. © 2003 KCWW. All rights reserved.

Manufactured by Ballard Medical Products, Draper, Utah 84020 USA
Distributed in the U.S. by Kimberly-Clark Global Sales, Inc. Roswell, GA. 30076 USA
Kimberly-Clark N.V., Belgicastraat 17, 1930 Zaventem, Belgium
<http://www.kchealthcare.com> In USA call 1-800-528-5591; International call + 1 801 572 6800